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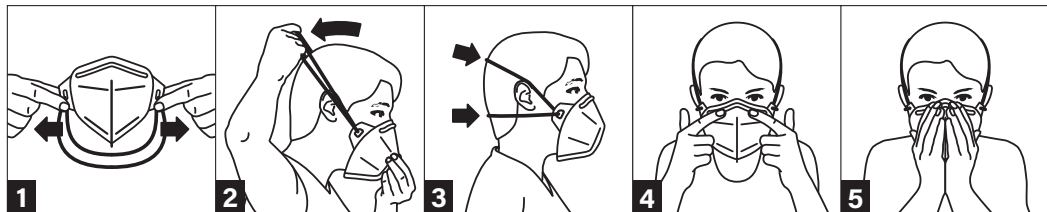
# CE 2834

**EN:** Before using the filtering half mask as a breathing apparatus, the following information must be read and subsequently followed.

**Use:**

- The choice of the most suitable personal protective equipment (PPE) depends on the respective situation and may only be made by qualified persons.
- This mask offers high wearing comfort and **protection of the respiratory tract against solid and liquid aerosols** in accordance with EN 149:2001+A1:2009.

**Checking before use / warning:**

- Before using the mask, check whether it is undamaged and the specified storage period has not been exceeded.
- The mask is not suitable for use in explosion hazard areas and does not protect against gases and vapours.
- The oxygen content of the respiratory air must be at least 19.5 % by volume.
- The mask must not be used if the concentration, nature or properties of the harmful substances are unknown.
- No changes must ever be made to the mask.

**Preparation:**

- Clean your hands
- Open the package and remove the mask
- Spectacle wearers: remove spectacles

**Apply and adjust the fit:**

- Open the mask and pull the head straps parallel
- Hold the mask over your nose, mouth and chin, lightly secure on the face with one hand on the outside, and with your free hand pull the two retaining straps over your head.
- Pull the lower retaining strap under your ears and pull the upper retaining strap over the back of your head
- Press the nose clip onto your face with both index fingers. Spectacle wearers: replace spectacles.

**Warning:**

- Verify that the mask fits properly and tightly. Breathe in sharply; if the mask does not seal properly, adjust the fit again. The mask should fit closely and cover your mouth and nose completely. If facial and/or beard hair is present below the sealing line, the effectiveness of the mask can no longer be guaranteed.

**Use:**

- The mask is intended for use during only one shift and must not be re-used.
- The mask may no longer be used:
  - if it is moistened by the respiratory air
  - if it is damaged / heavily contaminated or no longer closes tightly

**Removal and disposal of the mask:**

- Spectacle wearers: remove spectacles
- Hold the retaining straps with both hands, pull over your head and remove with the mask from your face
- Remove the mask without touching the mask component and discard in a closed waste container at the latest after the end of the shift
- Clean your hands
- Spectacle wearers: replace spectacles

**Storage and shelf life:**

- Storage and transport clean and dry between -5 °C to +25 °C and <80 % rel. atmospheric humidity. Store in the original packaging.
- The end of the storage period is printed on the packaging.

**Konformity:**

The particle filtering half mask meets the requirements of the European Regulation (EU) 2016/425. The type examination and the conformity assessment procedure were performed with the participation of the following accredited testing and certification body: CCQS Certification Services Ltd., Block 1, Blanchardstown Corporate Park, Dublin 15 D15 FWP6

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Download the EU Declaration of Conformity at  
[www.flawaconsumer.com/104910](http://www.flawaconsumer.com/104910)

V2/04.11.2020